Fast-Pack

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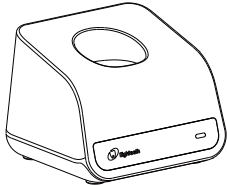
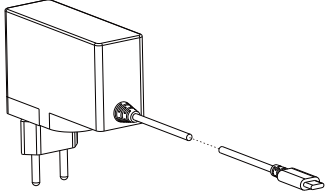
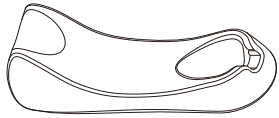
	лечение на мястото на употреба	<p>from the components, and put them in container for transportation.</p> <p>Prepare the components directly after treatment.</p> <p> WARNING</p> <p>Do not submerge the components or wipe them with any of the following functional water (acidic electrolyzed water, strong alkaline solution, or ozone water), medical agents (glutaral, etc.), or any other special types of water or commercial cleaning liquids. Such liquids may result in metal corrosion and adhesion of the residual medical agents to the components.</p>
<p>The following Step 2 to Step 4 are operated in a washer-disinfector:</p> <p> ВНИМАНИЕ</p> <ul style="list-style-type: none"> ● Use only approved washer-disinfectors according to EN ISO 15883, maintain and calibrate it regularly. ● Follow instructions and observe concentrations given by the manufacturer (see general recommendations). ● Sufficient rinsing step should be available in purified water (max 10 germs/ml and max 0.25 endotoxin units/ml) ● Avoid any contact between the cartridge and any instrument, kit, support or container. ● Make sure the components are dry before moving to the #5 step. 		
2	Cleaning: Automated	<p>Carefully put the components (Heating needle) into the washer-disinfector and set the parameters as follows:</p> <ul style="list-style-type: none"> - Pre-cleaning: water temperature <30°C, 2 min; - Cleaning: water temperature 45°C, 5 min; use an enzyme detergent solution (mild and aldehyde free solution) which is suitable to be used with washer-disinfector, and use in accordance with the IFU of the detergent solution manufacturer; - Rinsing: water temperature 45°C, 1 min (rinsing twice).
3	Disinfection: Thermal	<p>Thermal disinfection at least 5 min at 90°C/194° F, make sure A0 value ≥3000.</p> <p>Rinse with purified water at 70°C for 1 minute.</p>
4	Drying	<p>Heat: 20min, 90°C/194° F</p>

5	Maintenance and Inspection	Inspect components and sort out those with defects. Dirty components must be cleaned and disinfected again.
6	Packaging	<p>Pack each component in a separate steam-sterilization pouch.</p> <p> ВНИМАНИЕ</p> <ul style="list-style-type: none"> ● Check the validity period of pouch given by the manufacturer to determine the shelf life. ● Use pouches which resist to a temperature up to 141°C(286°F) and in accordance with EN ISO 11607.
7	Sterilization	<p>Steam sterilization at 134°C at least 6 minutes.</p> <p>Minimum drying time after sterilization: 10 minutes.</p> <p> ВНИМАНИЕ</p> <ul style="list-style-type: none"> ● Use only approved autoclave devices according to EN 13060 or EN 285. ● Use a validated sterilization procedure according to ISO 17665. ● Respect the maintenance procedure of the autoclave device given by the manufacturer. ● Use only this recommended sterilization procedure. ● Control the efficiency (packaging integrity, no humidity, color change of sterilization indicators, physicochemical integrators, digital records of cycles parameters). ● The sterilization procedure must comply with ISO 17665. ● Waiting for cooling before touching.
8	Storage	<p>Keep the components in sterilization packaging in a dry and clean environment.</p> <p> ВНИМАНИЕ</p> <ul style="list-style-type: none"> ● Sterility cannot be guaranteed if packaging is open, damaged or wet. ● Check the packaging before using it (packaging integrity, no humidity and validity period).
<p> NOTE</p> <p>Heating needle head is easy to damage by external force, take care during maintenance.</p>		

The instructions provided above have been validated by the manufacturer of the medical device as being capable of preparing a medical device for use. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process. Likewise, any deviation by the processor from the instructions provided should be properly evaluated for effectiveness and potential adverse consequences.

8.4 Disinfection components

Disinfection components		
Fast-Pack Handpiece 	Charge Base 	Adapter 
Handpiece Base 		
<p>Wipe the components with a piece of gauze that has been dampened with Ethanol for Disinfection (Ethanol 70 to 80 vol%) and wrung out thoroughly.</p>		
<p> ЗАБЕЛЕЖКА</p> <ul style="list-style-type: none"> ● Do not use anything except Ethanol for Disinfection (Ethanol 70 to 80 vol%). ● Do not use too much ethanol as it's going into machine and damage the components inside. <p>Do not spray any liquid directly on the handpiece. Do not allow any moisture to get into the handpiece.</p>		

9. Troubleshooting

When trouble is found, check the following points before contacting your distributor. If none of these are applicable or the trouble is not remedied even after action has been taken, the product may have failed. Contact your distributor.

Problem	Cause	Solution	Ref. chap
The power is not turned on.	The battery is flat.	Charge the battery.	7.1
	Press the power switch too short time.	Long press the power switch.	5
The power LED on charge base does not light.	Using a wrong adapter.	Use the original adapter.	4.3
	The adapter is not connected.	Check the connection.	4.3
	The plug of the adapter is not inserted into the outlet.	Check the connection.	/
	There is no electricity in the outlet.	Check the connection.	/
No charge indicator flash on handpiece screen	Put the handpiece into the charge base in the wrong direction.	Check the direction.	4.3
	Charge pin of charge base unable to rebound.	Remove debris which between move part and base of the charge pin.	/
	Contactors are dirty.	Cleaning the surface of contactors.	/
	The charge base broken.	Using adapter connect to handpiece directly, and Contact your distributor.	/
Handpiece screen does not appear	The handpiece broken.	Check if there is a sound of beep or motor, and Contact your distributor.	/
No sound.	Beep volume set to 0.	Set beep volume to 1, 2 or 3.	6.2

10. Technical Data

Manufacturer	Changzhou Sifary Medical Technology Co.,Ltd
Model	Fast-Pack
Dimensions	23cm x 17cm x 8cm \pm 1cm (Outer box)
Weight	1kg \pm 10%
Power supply	Lithium ion battery: 3.63V, 2600mAh, \pm 10%
Charger power supply	AC 100-240 V, \pm 10%
Charger power output	6V \cdot 3A
Frequency	50/60Hz, \pm 10%
Temperature	90°C~250°C
Electrical safety class	Class II
Applied part	B
Operating conditions	Use: in enclosed spaces Ambient temperature: 5°C / 40 °C Relative humidity: <80% Operating altitude < 2000m above sea level
Transport and storage conditions	Ambient temperature: -20 °C / +55 °C Relative humidity: 20% - 80 % Atmospheric pressure: 70kPa-106kPa

/ IEC 61000-4-4	± 2 kV ± 1 kV /		EUT 3 /
IEC 61000-4-5	± 1 kV I ± 2 kV		EUT
IEC 61000-4-11	<5% U_T (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 sec		EUT
- (50/60 Hz) IEC 61000-4-8	3 A/m	3 A/m	
U			

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All rights reserved.
Changzhou Sifary Medcial Technology Co., Ltd